



Revitalize



My Filler's "New Life to the Skin" widens its application horizons and creates **My Filler "Revitalize"**, the special line of products for Mesotherapy procedures.

My Filler Revitalize is a state-of-the-art range and complete portfolio of sterile topical cosmetics for professional use.

My FILLER Revitalize White Skin

Mesotherapy allows to reach the deeper layers of skin. Depositing the serum, **Revitalize White Skin**, increasing freshness and glow to the skin, thanks to its special formulation.

The effectiveness of whitening Meso is much higher than long term topical whitening creams.

REVITALIZE WHITE SKIN

Indications: Lighten skin tone and reduce skin spots.

Application: **Revitalize White Skin** protocol is recommended through laser, therapy with microneedle and/or virtual mesotherapy for electroporation.

1st and 2nd Month: 8 applications (once a week).

If the final results achieved before the 8 applications, the protocol can be suspended.

The process can speed up with special techniques (E.g.: Revitalize Booster).

Contents:

- Kojic Acid
- Retinol
- Ascorbic Acid (Vitamin C)

Packaging: 5 vials, 5 ml each.

Expiration: 3 years.



MYF505



Non injectable



Whitening



Sterile



External Use



Body




Face




MyFILLER

Revitalize White Skin

REVITALIZE PROTOCOL

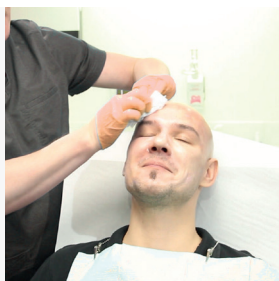


1. Clean, sterilise and applied anaesthetic topical cream to the chosen area to be treated.
2. After 30-60 minutes check if the area is numb. Remove topical cream with gauze and warm water. Exfoliate the area with a double cleanse, sterilise area and blot dry. Protect patient using appropriate means of covering like gauze, micropore tape and head-band.

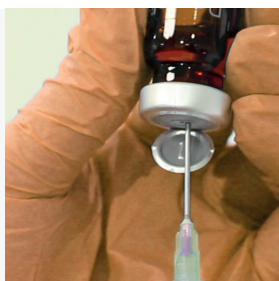

3. Select chosen mesotherapy product and draw down 1ml into sterile container.



4. Apply the product at area being treated with a cotton bud or gauze.
5. Choose depth on Revitalize Pen to maximum depth for the areas following (1mm face and 2-2.5mm body) and begin to stamp the mesotherapy product into the skin covering all areas.
6. Apply another layer of mesotherapy product.

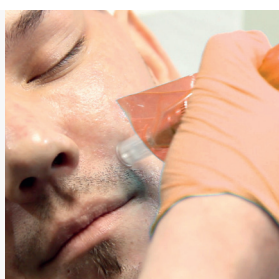

7. Reduce Revitalize pen depth to half previous chosen. Create rows vertical and horizontal to cover all the areas.
8. Apply another layer of mesotherapy and repeat using 's' sweeping motions with the Reduce Revitalize to cover all areas.
9. Apply the remaining mesotherapy product liberally all over with cotton bud or gauze.



10. A mask can be used to relax the skin for 5 minutes.
11. Blot the area dry with a tissue.
12. Apply moisturiser and/or SPF cream.